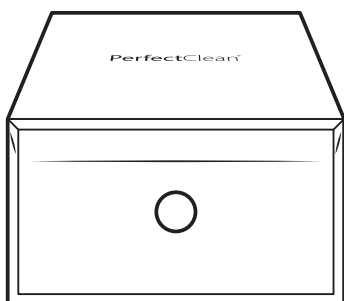
PerfectClean

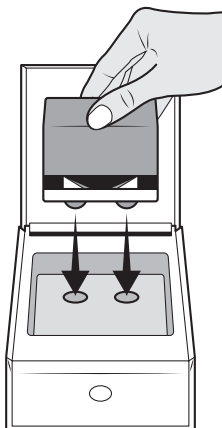
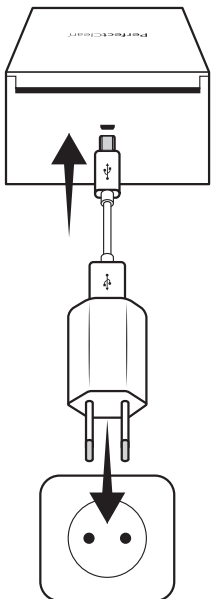
connexx

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connexx

Connexx is a trademark of Sivantos Group

1

Make sure the PerfectClean is flat, do not overturn it.

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EN

1 Plug in the PerfectClean. The blue indicator lights to show that it is powered on.

If no action occurs for 30 seconds, the indicator turns off.

Before the first use, make sure to refill the PerfectClean (completely empty the refill), otherwise the cycle will not start. For more information, please refer to the refill instructions.

2 Open the battery compartment (the batteries can remain in their housing), place the hearing aids into the box, placing the tips in the two holes provided. Close the cover.

3 Press the touch button once to start the cycle (the white flashing indicator means the request has been taken into account). The blue indicator flashes slowly, indicating that the box is running. The cycle will only run if the cover is closed. The PerfectClean stops automatically at the end of the cleaning cycle (1h).

The indicator turns off. Once the cycle has ended, rub the hearing aids with a clean absorbent cloth.

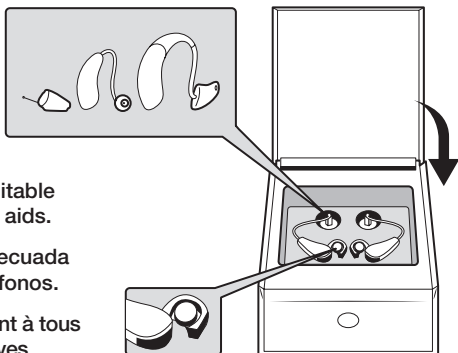
Technical data

Functions

Internal size (lxwxh): 71x18x87 mm
 External size (lxwxh): 106x68x122 mm
 Weight (g): 400g
 UV lamp: 253.7nm
 Max. power: 8,5W
 Standby power: 0.04w
 Software: ØØ PC-LOG-02.00
 Storage conditions from -20°C to +65°C

Dehumidification temperature: <48°C
 Cycle duration: 60 min.
 Operating temperature: From 12°C to 40°C


Power Supply:
 Input: 100-240Vac 50-60Hz
 Output: 5Vdc 10W mini
 Connectors: USB - micro USB cable

2

The PerfectClean is suitable for all types of hearing aids.

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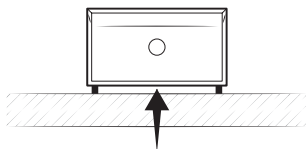Place PerfectClean on a smooth, flat surface.
Do not obstruct the air intake underneath PerfectClean.

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One refill can allow between 40 and 50 cycles*, between 20 and 25 cycles* when the tank is empty (for first use or after emptying it for transport).

*These quantities are given for your information only. They can vary depending on the hearing aids.

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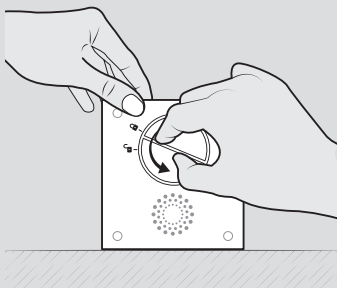
It is important to place the PerfectClean on its front side (with the start button) to prevent any fluid flow. Rotate the cap to the left, from the  position to the  position.

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Option - Opción - Option A

For a little dirty filter: clean it with warm water.

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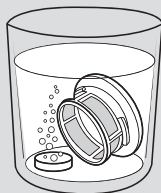
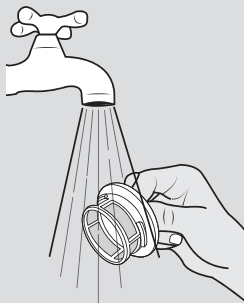
Option - Opción - Option B

For a very dirty filter: clean it with a cleaning tablet (20 minutes in a glass) and then rinse it with warm water.

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After cleaning it, replace the filter cap in the  position and turn it to the right, to the  position.

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Medical Device Directive

This product meets the requirements of directive 93/42/EEC on Medical Devices.

Function details

Cleaning

PerfectClean removes earwax from hearing aids thanks to a cleaning solution.

Dehumidification

The PerfectClean has a patented pulsed hot air dehumidification system

UV-C

PerfectClean has a short-wavelength UV-C lamp that optimises cleaning efficiency.

Handling instructions

Only refill the PerfectClean when the refill indicator is on. Use the full refill. Make sure the cover is closed before starting the cycle, otherwise the PerfectClean will not start. Do not open the cover if the PerfectClean is running. When the cycle is complete, the PerfectClean indicators turn off automatically. Place the PerfectClean on a flat, dry and stable surface. **Never tip or overturn the PerfectClean, even when it is not running.** Do not directly look at the UV-C lamp located in the dehumidification chamber. When disconnecting, pull the plug and not the cable.

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Emptying (to travel or transport)

1. Place the PerfectClean on its front face (the side with the start button).
2. Remove the filter cap by pivoting it to the left to the  position.
3. Put the PerfectClean flat over a sink to drain the remaining liquid.
4. Refit the cap in the  position and then turn it to the right to the  position.

In order to maintain optimum hearing aid quality, the daily use of the PerfectClean is recommended.

Safety instructions

Keep out of the reach of children.

Any work, other than cleaning and routine care by the client, must be carried out by the manufacturer's customer service or by an approved professional.

Do not place the PerfectClean, the power supply cable or the plug in water or any other liquid.

If the power supply cable or the plug are damaged, do not use the device and replace them immediately.

Replace your mains adapter with an original model.

Never use detergent or corrosive products to clean your PerfectClean.

Failure to follow these instructions will void the guarantee.

The manufacturer cannot be held liable for damage caused by incorrect use or handling.

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Product care advice

Clean using a soft cloth. Do not clean the PerfectClean using abrasive products. Do not spray liquids into the device.

Guarantee

24 months - (cable and transformer included). In the event of a malfunction, contact your hearing-aid specialist. Proof of purchase will be requested for all work by the customer service during the guarantee period. This guarantee does not apply to defects caused by an accident or incident or similar damage, by introducing a liquid, negligence, unsuitable use, dismantling of any kind, lack of care or any other circumstance caused by the user. Furthermore, this guarantee will not apply to damage caused by lightning or any other electric voltage variation. As a precaution, please disconnect the PerfectClean in the event of a thunderstorm.

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Identification data

This information can be found on the lower part of your PerfectClean.

Meaning of the abbreviations:

V Volts (Power supply voltage)

A Ampere (Power consumption)

 USB power supply symbol

 Indoor use

 The product is made from ABS with a recycling code of 7

 This product is compliant with all European Directives applicable to Medical Devices.

 **DISPOSAL**
Dispose of previous appliances ecologically. Old appliances contain recyclable materials that can be recovered. Therefore, please dispose of your old appliances using the appropriate waste disposal services.

 I protect the environment, I disconnect my PerfectClean once my hearing aids are clean!

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