

Customer: AUDINELL

Ref manual : 4 PD-02\_A04

Open format : Width 150 x Height 140

Closed format : Width 75 x Height 140

Pages: 4

Brand: AUDINELL

Date: 07\_10\_2021

Software used: In-Design CC 2021

#### COVER

Print type: Offset

Print media: Offset White paper 70 gr

Lamination: none

Protective varnish: none

#### Colors used:



Cyan



Magenta



Yellow



Black

#### INSIDE PAGES

Print type: Offset

Print media: Offset White paper 70 gr

Protective varnish: none

#### Colors used:



Cyan



Magenta




Yellow




Black

#### Open format

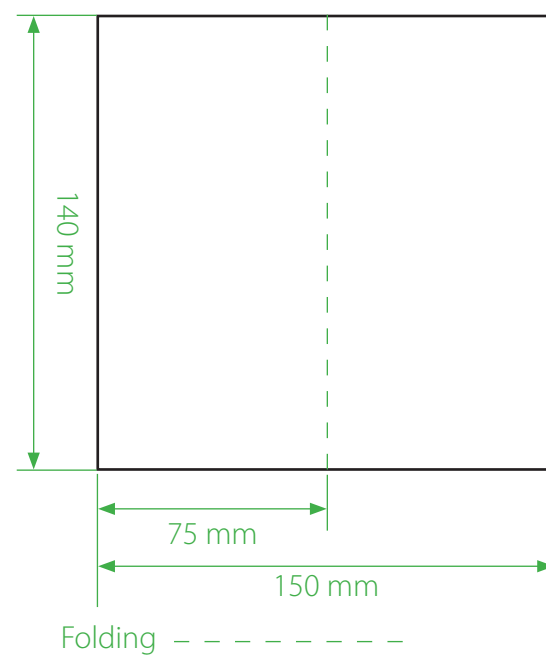



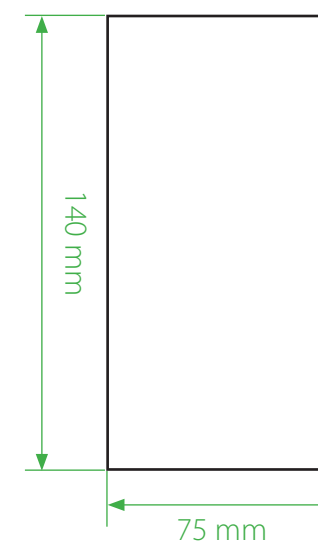



#### Closed format



# Identification data

Meaning of abbreviations:

 Manufacturer	 Medical device
 Indoor use	 The product is made from ABS with a recycling code of 7
 This product is compliant with Regulation (EU) 2017/745 applicable to medical devices	
 DISPOSAL - Dispose of previous appliances ecologically. Old appliances contain recyclable materials that can be recovered. Therefore, please dispose of your old appliances using the appropriate waste disposal service.	

# Technical data

## Characteristics

External dimensions: l x h x w (mm):

155x40x70

Internal dimensions: l x h x w (mm):

147x30x60

Weight: 145g

Power supply: 5Vdc / 0.4A

Max: 2W

Dehumidification temperature: < 42°C

Operating temperature: 12 to 40°C

Storage conditions: -20 to 65°C



## AC adapter

Input: 100-240Vac / 50-60Hz

Output: 5Vdc / 1A


# PerfectDry

Intensive dehumidification  
for hearing systems

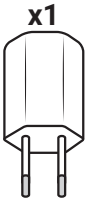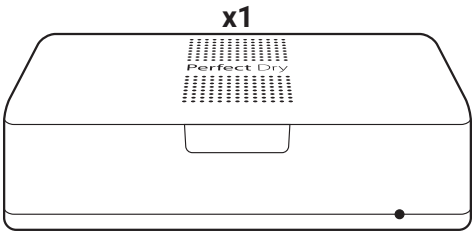


**1** Plug in PerfectDry. The light will come on briefly to show that the device is switched on.

**2** Clean your hearing aids with a cleansing wipe in order to remove impurities. Then open the battery section of your hearing aids (batteries may remain in hearing aids during drying). Put them in PerfectDry. Close the lid.

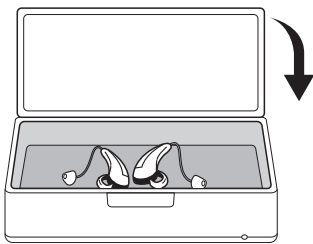

**3** Press to start the 2 hours 15 minutes cycle. PerfectDry will stop automatically at the end of the cycle (light goes off). The cycle may be interrupted at any time by simply pressing the switch.

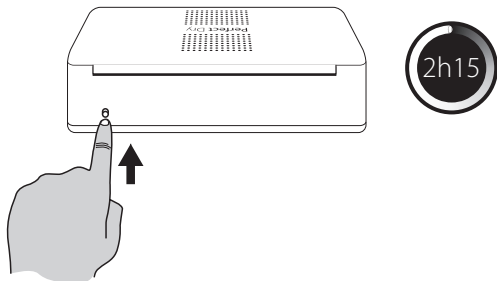

## Description

PerfectDry is a Medical Device offering the best way of drying any type of hearing aid. The drying system uses hot air circulated by natural convection.

Regular use of PerfectDry does not damage hearing aids (neither components nor resins).

## Safety instructions

Please read the Instructions carefully before using PerfectDry for the first time and keep them for future use: wrongful use releases the manufacturer from any responsibility.

Do not allow children to touch the device. Any service other than usual cleaning and maintenance must be carried out by the manufacturer's after-sales service or a qualified professional. Do not put PerfectDry, the electrical cable or plug in water or any other liquid. In case of damage to electrical cable or plug, do not use the device and change faulty parts immediately. Replace the AC adapter with an original model only. Never use detergents or corrosive products to clean PerfectDry.

Failure to follow the safety instructions will render the guarantee null and void. The manufacturer may not be held responsible for any damage due to incorrect use or handling.

All serious incidents related to the device should be the subject of a notification to the manufacturer and the competent authorities in the Member State in which the user is located.

## Regulation applicable to Medical devices

This product meets the requirements of Regulation (EU) 2017/745 covering medical devices.

## Guarantee

PerfectDry is guaranteed for 24 months from the date of purchase. In case of breakdown, please contact your hearing aid specialist. Proof of purchase will be required by the after-sales service team during the guarantee period. The guarantee will not apply in case of breakdown due to an accident or incident or related damage, introduction of liquid, negligence, wrongful use, if the material is disassembled, or in case of a lack of maintenance by the user. Moreover, the guarantee will not apply in case of faults caused by lightening or other variations of electric voltage. As a precautionary measure, we recommend that you unplug PerfectDry during storms.



I protect the environment, I disconnect my PerfectDry after the cycle!