**DEX™**  
A WIRELESS WORLD

**CALL-DEX**


 **WIDEX A/S** Nymoellevej 6, 3540 Lyngby, Denmark  
[www.widex.com](http://www.widex.com)

 **Manual no.:**  
9 514 0295 099 #02  
**Issue:**  
2015-08


  
9 514 029 50990 7

**WIDEX®**  
HIGH DEFINITION HEARING


1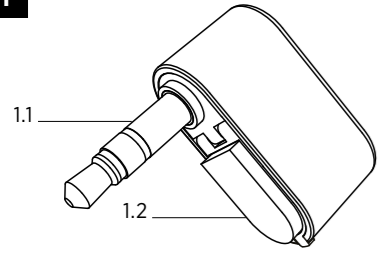

2

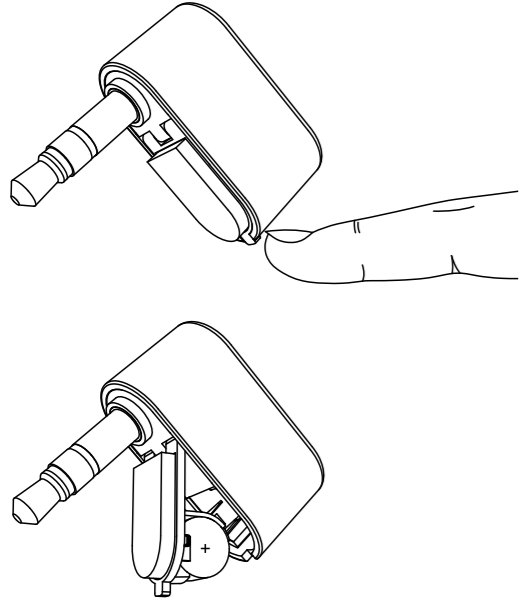3

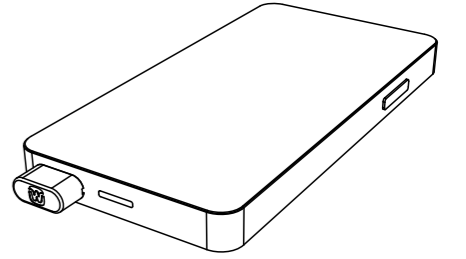4

UK	USER INSTRUCTIONS.....	3
DE	BEDIENUNGSANLEITUNG.....	12
FR	MODE D'EMPLOI.....	21
ES	INSTRUCCIONES DE USO.....	30
PT	INSTRUÇÕES DE UTILIZAÇÃO.....	39
IT	LIBRETTO D'ISTRUZIONI.....	48
NL	GEBRUIKSAANWIJZING.....	57
DK	BRUGSANVISNING.....	66
NO	BRUKSANVISNING.....	75
SE	BRUKSANVISNING.....	84
FI	KÄYTTÖOHJE.....	93
KR	사용 지침.....	102
JP	取扱説明書.....	111
CN	用户使用手册.....	121
TH	คู่มือการใช้งาน.....	130
CZ	NÁVOD K POUŽITÍ.....	139
SK	NÁVOD NA POUŽITIE.....	148
HU	HASZNÁLATI UTASÍTÁS.....	157
HR	KORISNIČKE UPUTE.....	166
CS	UPUTSTVO ZA UPOTREBU.....	175
SI	NAVODILA ZA UPORABO.....	184
BG	УПЪТВАНЕ ЗА УПОТРЕБА.....	193
GR	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ.....	202
TR	KULLANMA KILAVUZU.....	211
PL	INSTRUKCJA OBSŁUGI.....	220

LT	NAUDOJIMO INSTRUKCIJOS. ....	230
RU	ИНСТРУКЦИЯ ПО ЭКСПЛУАТАЦИИ. ....	239
RO	NSTRUCTIUNI DE UTILIZARE ....	248
EE	KASUTUSJUHEND ....	257
EG	مدخست سمل ا تاميل عت . ....	266
BO	KORISNIČKO UPUTSTVO ....	275
PK	فراص ے ئارب تا اادہ . ....	284
US/CA	USER INSTRUCTIONS. ....	293
LV	LIETOTĀJA INSTRUKCIJA ....	311
AL	UDHËZIMET E PËRDORIMIT ....	320
IR	ربراک ا امنهار . ....	329
VN	HƯỚNG DẪN SỬ DỤNG ....	338
TW	使用說明. ....	347
IL	שמתשמל תוארה . ....	356
PH	MGA PAGTUTURO PARA SA USER. ....	365
KZ	ПАЙДАЛАНУШЫ НҰҚҚАУЛАРЫ . ....	375
MY	ARAHAN PENGGUNA . ....	384
UA	ІНСТРУКЦІЯ З ЕКСПЛУАТАЦІЇ . ....	393
AFG	USER INSTRUCTIONS (PASHTO + DARI). ....	402
	FCC AND IC STATEMENTS. ....	420

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### Intended use

The device is intended to be plugged into a mobile phone and to stream the sound from the mobile phone directly to the hearing aids.

**Note:** The device uses a standard jack plug which is common in newer mobile phones. If your mobile phone uses a different configuration, it may not work with this product.

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To obtain replacement batteries, please consult your hearing care professional. It is important to take note of the expiry date and the recommendations on the battery pack regarding disposal of used batteries.

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## **HOW TO USE** – See illustrations 3 and 4

- Connect the jack plug to your mobile phone as illustrated.
- Make sure that the device is positioned so that it is flush with the edge of your phone. The device is now ready for use.
- When having a phone conversation, hold your phone close to your ear as you would normally do.

**NOTE:** We recommend you set the ringtone on your mobile phone to maximum.

**NOTE:** If your mobile phone supports Near Field Communication (NFC), we recommend that you turn off this feature.

## **MAINTENANCE**

- Clean with a dry cloth. Do not use chemicals.
- Do not expose to extreme temperatures or high humidity.
- Do not immerse in liquids.
- When the device is not in use keep it in a dry location out of reach of children and pets.
- Remove the battery if the device will not be used for several days.
- Never try to open or repair the device yourself. This may only be done by authorised personnel.

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## TROUBLESHOOTING

Problem	Possible cause	Solution
The CALL-DEX does not work	The device is not correctly connected to your mobile phone	Make sure the device is correctly connected
	The CALL-DEX battery is low or dead	Change battery
	The battery is not correctly inserted	Turn the battery over
	Out of operating range	Move within operating range
	The mobile phone is set to silent mode	Set to normal mode
	The hearing aid battery is low	Change hearing aid battery

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## WARNINGS

### WARNING

- Swallowing or improper use of the device or batteries can result in severe injury or even fatalities. In case of ingestion, contact a physician immediately.
- Batteries are very small and can easily be mistaken for pills or the like. Never put a battery or the device in your mouth for any reason as you may risk swallowing it.
- Risk of explosion if battery is replaced by an incorrect type or recharged. Dispose of used batteries according to the instructions.
- Do not carry your device with you if you are having an X-ray or MRI scan, or other scans or radiation treatments and never place it in a microwave oven.
- Keep the device and its parts, accessories and batteries out of reach of children and anyone else who might swallow such items or otherwise cause injury to themselves. Do not change batteries in front of them and do not let them see where you keep your battery supply.
- Do not use the device on aircraft or in hospitals without permission.
- Do not use the device in mines or other areas with explosive gases.
- Your device has been tested for interference according to international standards. Nevertheless, it is possible that unforeseen interference may occur in the hearing aid due to electromagnetic radiation from other products such as alarm systems, room surveillance equipment and mobile phones.
- Although your device has been designed to comply with the most stringent international electromagnetic compatibility standards, the possibility cannot be excluded that it may cause interference with other equipment, such as medical devices.
- When the device is connected to external mains-operated equipment such as a PC, audio source or similar, this equipment must comply with IEC 60065, IEC 60950-1, IEC 60601-1 or equivalent safety standards.

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In order to show caution, we advise to follow the guidelines recommended by manufacturers of defibrillators and pacemakers regarding use of mobile phones:

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Symbol	Title/Description
	<b>Manufacturer</b> The product is produced by the manufacturer whose name and address are stated next to the symbol. If appropriate, the date of manufacture may also be stated.
	<b>Date of manufacture</b> The date when the product was manufactured.
	<b>Use-by date</b> The date after which the product is not to be used.
	<b>Batch code</b> The product's batch code (lot or batch identification).
	<b>Catalogue number</b> The product's catalogue (item) number.
	<b>Serial number</b> The product's serial number.*
	<b>Keep away from sunlight</b> The product must be protected from light sources and/or The product must be kept away from heat

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Symbol	Title/Description
	<p><b>CE mark</b> The product is in conformity with the requirements set out in European CE marking directives.</p>
	<p><b>Alert</b> The product is identified by R&amp;TTE Directive 1999/5/EC as an equipment Class 2 product with some restrictions on use in some CE member states.</p>
	<p><b>RCM mark</b> The product complies with electrical safety, EMC and radio spectrum regulatory requirements for products supplied to the Australian or New Zealand market.</p>
	<p><b>Interference</b> Electromagnetic interference may occur in the vicinity of the product.</p>

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## DESCRIPTION OF DEVICE – See illustration 1

- 1.1 Jack plug
- 1.2 Battery lid

### WARNING

Read these instructions carefully before you start using the device.

### Intended use

The device is intended to be plugged into a cell phone and to stream the sound from the cell phone directly to the hearing aids.

**Note:** The device uses a standard jack plug which is common in newer cell phones. If your cell phone uses a different configuration, it may not work with this product.

## THE BATTERY

Your device uses a type 10 battery. We recommend zinc-air batteries.

To obtain replacement batteries, please consult your hearing care professional. It is important to take note of the expiration date and the recommendations on the battery pack regarding disposal of used batteries.

### Inserting the battery – see illustration 2

- Before inserting a new battery into the device, remove the adhesive tab. Once the tab has been removed, the battery will start functioning after a few seconds.
- Use the nail grip to gently open the battery lid as illustrated.
- Place the battery in the compartment so that the small plus (+) sign on the battery is upwards.
- Close the battery lid. If it does not close easily, the battery is incorrectly inserted.

**NOTE:** If you use the phone for conversations for more than 3 hours a day, we recommend changing the battery once a week.

If you use it less than 3 hours a day, we recommend changing the battery once every second week.

## WARNING

- Never leave an exhausted battery in the device. Exhausted batteries may leak, damaging the device.
- Do not use batteries if there is a sticky residue from the tab or other unwanted substance, as this can cause the device to malfunction.

## HOW TO USE – See illustrations 3 and 4

- Connect the jack plug to your cell phone as illustrated.
- Make sure that the device is positioned so that it is flush with the edge of your phone. The device is now ready for use.
- When having a phone conversation, hold your phone close to your ear as you would normally do.

**NOTE:** We recommend you set the ringtone on your cell phone to maximum.

**NOTE:** If your cell phone supports Near Field Communication (NFC), we recommend that you turn off this feature.

## MAINTENANCE

- Clean with a dry cloth. Do not use chemicals.
- Do not expose to extreme temperatures or high humidity.
- Do not immerse in liquids.
- When the device is not in use keep it in a dry location out of reach of children and pets.
- Remove the battery if the device will not be used for several days.
- Never try to open or repair the device yourself. This may only be done by authorized personnel.

## INFORMATION ABOUT EMISSION

The emission from CALL-DEX like all Widex radio applications is tested by an accredited laboratory.

Unwanted emissions and therefore also risk of interference with other devices are negligible and have been found to be within the very strict requirements outlined in the international specifications for this area.

Widex is obliged to document these internal test reports in order to be able to obtain certification in compliance with both the Medical Devices Directive in the EU and FCC regulations in the USA and Canada.

For further information especially regarding patients with medical implants, please download the document: "Information on WidexLink™" at: <http://widex.com/widexlink>

## TROUBLESHOOTING

Problem	Possible cause	Solution
The CALL-DEX does not work	The device is not correctly connected to your cell phone	Make sure the device is correctly connected
	The CALL-DEX battery is low or dead	Change battery
	The battery is not correctly inserted	Turn the battery over
	Out of operating range	Move within operating range
	The cell phone is set to silent mode	Set to normal mode
	The hearing aid battery is low	Change hearing aid battery

If the problems persist, contact your hearing care professional.

## WARNINGS

### WARNING

- Swallowing or improper use of the device or batteries can result in severe injury or even fatalities. In case of ingestion, contact a physician immediately.
- Batteries are very small and can easily be mistaken for pills or the like. Never put a battery or the device in your mouth for any reason as you may risk swallowing it.
- Risk of explosion if battery is replaced by an incorrect type or recharged. Dispose of used batteries according to the instructions.
- Do not carry your device with you if you are having an X-ray or MRI scan, or other scans or radiation treatments and never place it in a microwave oven.
- Keep the device and its parts, accessories and batteries out of reach of children and anyone else who might swallow such items or otherwise cause injury to themselves. Do not change batteries in front of them and do not let them see where you keep your battery supply.
- Do not use the device on aircraft or in hospitals without permission.
- Do not use the device in mines or other areas with explosive gases.
- Your device has been tested for interference according to international standards. Nevertheless, it is possible that unforeseen interference may occur in the hearing aid due to electromagnetic radiation from other products such as alarm systems, room surveillance equipment and cell phones.
- Although your device has been designed to comply with the most stringent international electromagnetic compatibility standards, the possibility cannot be excluded that it may cause interference with other equipment, such as medical devices.
- When the device is connected to external mains-operated equipment such as a PC, audio source or similar, this equipment must comply with IEC 60065, IEC 60950-1, IEC 60601-1 or equivalent safety standards.

- Interference with active Implants.

In order to show caution, we advise to follow the guidelines recommended by manufacturers of defibrillators and pacemakers regarding use of cell phones:

If you wear an active implantable device keep the Wireless Hearing Aids and Hearing Aid Accessories such as wireless remote controls or communicators at least 15 cm away from the implant.

If you experience any interference, do not use the hearing aids and/or hearing aid accessories, and contact the manufacturer of the implant. Please note that interference can also be caused by power lines, electrostatic discharge, airport metal detectors etc.

If you have an active brain implant, please contact the manufacturer of the implant for risk evaluation.

If you have an implantable device, we advise keeping magnets\* at least 15 cm away from the implant. (\*= can be specified as Autophone magnet, hearing instrument case, magnet in a tool, etc.)

## DIRECTIVE 1999/5/EC

Hereby, Widex A/S declares that this CALL-DEX is in compliance with the essential requirements and other relevant provisions of Directive 1999/5/EC.

A copy of the Declaration of Conformity according to 1999/5/EC can be found at: <http://www.widex.com/doc>

N26346

Electrical and electronic equipment (EEE) contains materials, components and substances that can be hazardous and present a risk to human health and the environment when waste electrical and electronic equipment (WEEE) is not handled correctly.

Do not dispose of hearing aids, hearing aid accessories and batteries with ordinary household waste.

Hearing aids, batteries and hearing aid accessories should be disposed of at sites intended for waste electrical and electronic equipment, or given to your hearing care professional for safe disposal. Proper disposal helps to protect human health and the environment.

# SYMBOLS

Symbols commonly used by Widex A/S in medical device labelling (labels/IFU/etc.)

Symbol	Title/Description
	<b>Manufacturer</b> The product is produced by the manufacturer whose name and address are stated next to the symbol. If appropriate, the date of manufacture may also be stated.
	<b>Date of manufacture</b> The date when the product was manufactured.
	<b>Use-by date</b> The date after which the product is not to be used.
	<b>Batch code</b> The product's batch code (lot or batch identification).
	<b>Catalog number</b> The product's catalog (item) number.
	<b>Serial number</b> The product's serial number.*
	<b>Keep away from sunlight</b> The product must be protected from light sources and/or The product must be kept away from heat

Symbol	Title/Description
	<p><b>Keep dry</b> The product must be protected from moisture and/or The product must be kept away from rain</p>
	<p><b>Lower limit of temperature</b> The lowest temperature to which the product can be safely exposed.</p>
	<p><b>Upper limit of temperature</b> The highest temperature to which the product can be safely exposed.</p>
	<p><b>Temperature limits</b> The highest and lowest temperatures to which the prod- uct can be safely exposed.</p>
	<p><b>Consult instructions for use</b> The user instructions contain important cautionary infor- mation (warnings/precautions) and must be read before using the product.</p>
	<p><b>Caution/Warning</b> Text marked with a caution/warning symbol must be read before using the product.</p>
	<p><b>WEEE mark</b> <b>“Not for general waste”</b> When the product is to be discarded, it must be sent to a designated collection point for recycling and recovery.</p>

Symbol	Title/Description
	<b>CE mark</b> The product is in conformity with the requirements set out in European CE marking directives.
	<b>Alert</b> The product is identified by R&TTE Directive 1999/5/EC as an equipment Class 2 product with some restrictions on use in some CE member states.
	<b>RCM mark</b> The product complies with EMC and radio spectrum regulatory requirements for products supplied to the Australian or New Zealand market.
	<b>Interference</b> Electromagnetic interference may occur in the vicinity of the product.

\*The six- or seven-digit number on the product is the serial number. Serial numbers may not always be preceded by **SN**

## REGULATORY INFORMATION

### Radio transmitter/cables/transducers

The CALL-DEX contains a radio transmitter/receiver with the following.

Radio transmitter parameters:

- Frequency (range): 10,6 MHz (10,2 – 11,0 MHz)
- Bandwidth (-15dB): 660 kHz
- Channel: Single channel radio
- Modulation: FSK
- Radiated output power: x.x nW / -yy dBm
- Magnetic field strength: -zz dB $\mu$ A/m @ 10 m
- Duty Cycle: Up to 100 % (averaged over 1 hour of operation)
- Simplex capability

### Cables and transducers:

No Cables or transducers are used.

## Guidance and manufacturer's declaration

### Electromagnetic emissions

The CALL-DEX is intended for use in the electromagnetic environment specified below. The customer or the user of a CALL-DEX should assure that it is used in such an environment.

<b>Emissions test</b>	<b>Compliance</b>	<b>Electromagnetic environment - guidance</b>
RF emissions CISPR 11	Group 1	The CALL-DEX uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The CALL-DEX is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable *)	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Not applicable *)	

\*) *Battery powered equipment*

## Electromagnetic immunity

The CALL-DEX is intended for use in the electromagnetic environment specified below. The customer or the user of a CALL-DEX should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test level	Compliance level	Electromagnetic environment - guidance
Electro-static discharge (ESD) IEC 61000-4-4	$\pm 6$ kV contact $\pm 8$ kV air	$\pm 6$ kV contact $\pm 8$ kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transients/ burst IEC 61000-4-5	$\pm 2$ kV for power line supplies $\pm 1$ kV for input/ output lines	Not applicable *)  Not applicable *)	Not applicable *)
Surge IEC 61000-4-5	$\pm 1$ kV line(s) to line(s) $\pm 2$ kV line(s) to earth	Not applicable *)  Not applicable *)	Not applicable *)

<p>Voltage dips, short interruptions and voltage variations on power supply input lines</p> <p>IEC 61000-4-11</p>	<p>&lt;5 % <math>U_T</math> (&gt;95 % dip in <math>U_T</math>) for 0.5 cycle</p> <p>40 % <math>U_T</math> (60 % dip in <math>U_T</math>) for 5 cycles</p> <p>70 % <math>U_T</math> (30 % dip in <math>U_T</math>) for 25 cycles</p> <p>&lt;5 % <math>U_T</math> (&gt;95 % dip in <math>U_T</math>) for 5 s</p>	<p>Not applicable *)</p>	<p>Not applicable *)</p>
<p>Power frequency (50/60 Hz) magnetic field</p> <p>IEC 61000-4-8</p>	<p>3 A/m</p>	<p>3 A/m</p>	<p>Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment</p>

NOTE  $U_T$  is the a.c. mains voltage prior to the application of the test level.

\*) *Battery powered equipment*

### Electromagnetic immunity – cont.

The CALL-DEX is intended for use in the electromagnetic environment specified below. The customer or the user of a CALL-DEX should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test level	Compliance level	Electromagnetic environment – guidance
Conducted RF  IEC 61000-4-6	3 Vrms  150 kHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the CALL-DEX, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.  <b>Recommended separation distance</b>  $d = 1.2 \sqrt{P}$

Radiated RF	3 V/m	3 V/m	<p data-bbox="646 115 899 179"> <math>d = 1.2 \sqrt{P}</math>  80 MHz to 800 MHz </p> <p data-bbox="646 249 899 313"> <math>d = 2.3 \sqrt{P}</math>  800 MHz to 2.5 GHz </p> <p data-bbox="594 391 951 609"> Where <math>P</math> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <math>d</math> is the recommended separation distance in metres (m). </p> <p data-bbox="588 684 957 872"> Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey <sup>a</sup>, should be less than the compliance level in each frequency range <sup>b</sup>. </p> <p data-bbox="588 896 957 988"> Interference may occur in the vicinity of equipment marked with the following symbol: </p> <div data-bbox="708 1069 832 1184" style="text-align: center;">  </div>
-------------	-------	-------	--

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the CALL-DEX is used exceeds the applicable RF compliance level above, the CALL-DEX should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or re-locating the CALL-DEX.

b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

## Recommended separation distances

### Recommended separation distances between portable and mobile RF communication equipment and the CALL-DEX.

The CALL-DEX is intended for use in the electromagnetic environment in which RF disturbances are controlled. The customer or the user of the CALL-DEX can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the CALL-DEXs as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

---

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

---

This CALL-DEX may be interfered with by other equipment even if that other equipment complies with CISPR emission requirements.

## EMC/EMI Compliance

The CALL-DEX comply with the following EMC/EMI standards:

<b>Standard</b>	<b>Test type</b>	<b>Note</b>
47 CFR Part 15, subpart C	RF emissions	USA Federal Communications Commission (FCC) requirements to intentional radiators.
EN 300 330-2 V1.5.1	RF emissions incl. Spurious emission	EMC and radio spectrum matters for Short Range Devices in the frequency range 9 kHz – 25 MHz
IEC 60601-1-2:2007	EMC emission Immunity, RF and ESD	Medical electrical equipment. General requirements for basic safety and essential performance. Electromagnetic compatibility.
EN 301 489-3 V1.4.1	Immunity, RF and ESD	Standard for Low Power Transmitters in the frequency range 9 kHz – 40 GHz

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**FCC ID: TTY-CDEX**

**IC: 5676B-CDEX**

### **Federal Communications Commission Statement**

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

#### **NOTE:**

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

#### **NOTE:**

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Changes or modifications to the equipment not expressly approved by Widex could void the user's authority to operate the equipment.

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Under Industry Canada regulations, this radio transmitter may only operate using an antenna of a type and maximum (or lesser) gain approved for the transmitter by Industry Canada.

To reduce potential radio interference to other users, the antenna type and its gain should be so chosen that the equivalent isotropically radiated power (e.i.r.p.) is not more than that necessary for successful communication.

This device complies with Industry Canada licence-exempt RSS standard(s). Operation is subject to the following two conditions:

- (1) this device may not cause interference, and
- (2) this device must accept any interference, including interference that may cause undesired operation of the device.

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