WIDEX A/S

Nymoellevej 6, DK-3540 Lyngby, Denmark
<https://global.widex.com>

Manual no.:
9 514 0380 080 #04
Issue:
2020-01


DEX™

**COM-DEX
Remote Mic**


GB
DE
FR
ES
DK
IT
JP
US

WIDEX®
HIGH DEFINITION HEARING


1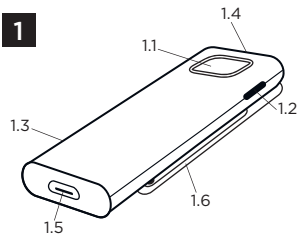


2

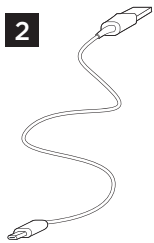
3

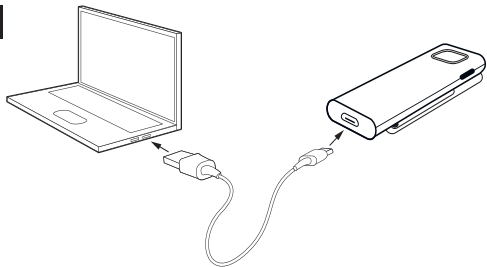4

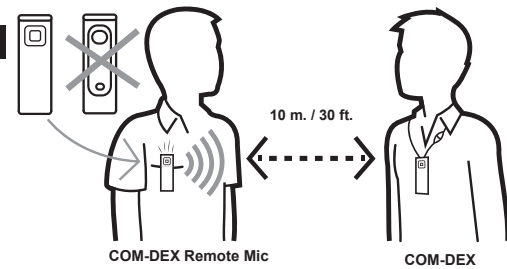5

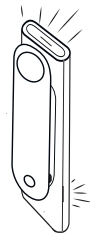GB	USER INSTRUCTIONS	4
DE	BEDIENUNGSANLEITUNG	14
FR	MODE D'EMPLOI	24
ES	INSTRUCCIONES DE USO	34
DK	BRUGSANVISNING	44
IT	LIBRETTO D'ISTRUZIONI	54
JP	取扱説明書	64
US	USER INSTRUCTIONS	74
	FCC AND ISSED STATEMENTS	100

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Visual indicators - see illustration 5

Your device has two LEDs that flash with different colours. The purpose of these colours is to inform you of which mode your device is in at any given time.

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Your device should be stored and transported within the temperature and humidity ranges of -10° to +50°C (14°F to 122°F) and <90% RH.

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N26346

Information regarding disposal

Do not dispose of hearing aids, hearing aid accessories and batteries along with ordinary household waste.

Hearing aids, batteries and hearing aid accessories should be disposed of at sites intended for electrical waste and electronic equipment, or given to your hearing care professional for safe disposal.

SYMBOLS

Symbols commonly used by Widex A/S in medical device labelling (labels/IFU/ etc.)

Symbol	Title/Description
--------	-------------------

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DESCRIPTION OF DEVICE - see illustration 1

1.1 Streaming button

1.2 Power button

1.3 LED

1.4 Microphone and LED

1.5 Charging port

1.6 Clip

Intended use

The COM-DEX Remote Mic device is a microphone with an integrated radio intended for transmitting sound to a COM-DEX connected to the user's hearing aid(s). The aim is to improve the hearing aid user's ability to hear speech in difficult listening situations. **For optimal connectivity the device and the COM-DEX must be within sight of each other.**

NOTE

Your Remote Mic is only intended to be used with a COM-DEX (with firmware V1.3 or later).

Accessories - see illustration 2

USB cable

USING YOUR DEVICE

Charging your device - see illustration 3

Connect the USB cable to your device and the power supply. It takes approximately 3 hours to charge your device.

NOTE

Before you can use the device, you must disconnect the USB cable.

Battery life: 8 hours streaming.

Turning the device on and off

To turn on your device:

- Press the power button and keep it pressed for 3 seconds.

To turn off your device:

- Press the power button and keep it pressed for 3 seconds.

The device turns off automatically if it has not been streaming sound for more than 30 minutes.

NOTE

To achieve the best sound possible, always turn your COM-DEX Remote Mic off when not in use.

Pairing your device

Pairing your device to your COM-DEX

To pair your device to your COM-DEX for the first time, follow these steps:

1. Make sure that your COM-DEX is in pairing mode.
2. Turn on your device. The first time you do this, the device will be set in pairing mode and it will automatically connect to your COM-DEX. This can take a few minutes.

NOTE

Once you have paired your device to your COM-DEX, it will remain so and will connect automatically if your COM-DEX is on.

Resetting the COM-DEX Remote Mic for a new pairing:

1. Turn on your device.
2. Reset your device to factory settings by pushing both buttons for 5 seconds. The device will be set to pairing mode.
3. Follow the pairing procedure described above.

Using the device - see illustration 4

Remove the protective film before using your device.

Place the device on the shirt collar of the person you'd like to hear.

Turn on the device. The Remote Mic starts streaming automatically.

NOTE

While streaming, your hearing aid microphones will be turned off. Once the Remote Mic stops streaming, the hearing aid microphones will be turned on again after a few seconds. If you want to turn on your hearing aid microphones while streaming, simply press the Room Off button on your COM-DEX.

To start and stop streaming, simply push the streaming button on your Remote Mic. While streaming, the LED will show a green light.

NOTE

If you want to operate your device via the COM-DEX app, you'll need to update the app.

Sound adjustment

The sound that is streamed from the device to COM-DEX can be adjusted in 3 levels. Simply push the power button on the side of the device and the sound level will be altered in a three step circle (highest level – lowest level – back to normal level).

Visual indicators - see illustration 5

Your device has two LEDs that flash with different colors. The purpose of these colors is to inform you of which mode your device is in at any given time.

- Constant red light on the **bottom** LED: The device is charging.
- Constant green light on the **bottom** LED: The device is fully charged.
- Flashing blue-red-blue-red light on **both** LEDs: The device is ready to be paired.

- Flashing blue light on the **top** LED: The device is on, but there is no connection to the paired COM-DEX.
- Flashing green light on the **top** LED: The device is on and connected to the COM-DEX, but it is not streaming.
- Constant green light on the **top** LED: The device is on and streaming.
- Occasionally flashing red light on **both** LEDs: The battery is low.

Maintenance

- Clean the device with a damp cloth. Do not use any liquids or chemicals and do not immerse in liquids.
- When the device is not in use keep it in a dry location out of reach of children and pets.
- Never try to open or repair the device yourself. This may only be done by authorised technicians.

TROUBLESHOOTING

Problem	Possible cause	Solution
The device does not work	1. Remote Mic battery low	1. Charge Remote Mic battery
	2. COM-DEX battery low	2. Charge your COM-DEX
	3. Out of range	3. Keep the devices in sight of each other and in range (up to 10m)
No sound	1. Not paired to COM-DEX	1. Repeat the pairing process
	2. COM-DEX is out of the hearing aids' range	2. Move into range
	3. Hearing aid batteries are low	3. Change the hearing aids' batteries

If the problem persists, contact your hearing care professional. For more information, visit: <https://global.widex.com>.

Warnings

Do not leave the device unattended while charging.

Your device should be stored and transported within the temperature and humidity ranges of -10° to $+50^{\circ}\text{C}$ (14°F to 122°F) and $<90\%$ RH.

Your device is designed to operate from 5°C (41°F) to 40°C (122°F).

Keep the device protected from dust or external particles.

We recommend using a 5V up to 500 mA USB adaptor.

Do not use the device in mines or other areas with explosive gases.

When the device is connected via micro USB cable to external mains-operated equipment such as a charger, PC, audio source or similar, this equipment must comply with IEC 60950-1, IEC 62368-1, IEC 60601-1 or equivalent safety standards.

Only the accessories and cable that are included in the sales box must be used. Using cables other than those specified or provided may result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

REGULATORY INFORMATION

Wireless functions and operating characteristics

The following tables summarizes the technical details of the wireless technology:

Technical details	Inductive radio	Bluetooth LE
Antenna type	Integral coil antenna	Integral frame antenna
Antenna dimensions	Ø 2 mm, L 5 mm	18 mm x 4.5 mm
Modulation	FSK	GFSK / FHSS
Magnetic field strength	-54 dB μ A/m @ 10 m	-
Output power (EIRP)	-	0 dBm / 1 mW
Range	0.3 m	6 m
Center frequency	10.6 MHz	2401 MHz – 2480 MHz
Channel	Single Channel	40
Bandwidth	660 kHz	2 MHz
Data rate	212 kbps	1 Mbps
Data flow	Simplex/Semi duplex	Duplex
Protocol	Random Access / Proprietary	BLE

Wireless security measures

Security of the wireless signals is assured through device system design that includes:

- Individual MAC address for each unit which is checked during each transmission.
- A built-in pairing table which specifies valid and legitimate pairing among units
- A proprietary Widex communication protocol which checks the package numbers during each transmission.
- A Cyclic Redundancy Check (CRC) to check data validity and correct errors.

Electromagnetic emissions/immunity standards

The product complies with the following EMC/EMI standards:

Standard	Test type	Note
47 CFR Part 15, Subpart B	RF emissions	US FCC requirements for unintentional radiators
47 CFR Part 18	RF emissions	US FCC requirements for Industrial, Scientific and Medical Equipment
IEC 60601-1-2	General EMC including RF emissions, RF immunity, ESD	Medical Electrical Equipment, Electro-magnetic compatibility

The following Table summarizes the technical details of the Bluetooth and WidexLink technology as it is implemented in the COM-DEX Remote Mic.

Unit	COM-DEX Remote Mic	Hearing aid
Type	Bluetooth*	W-link
Antenna type	Embedded ceramic antenna	Inductive antenna
Antenna dimensions	NA	Ø1.8 mm L – 4.85mm
Modulation	FHSS/GFSK, π/4 DPSK, 8DPSK	FSK
Magnetic Field Strength	NA	-54dBμA/m at 10m distance
Output power	2.5 mW EIRP +4dBm	29pW EIRP** -75 dBm
Range	<10m between COM-DEX Remote Mic and Bluetooth device	<1m remote unit to hearing aid <30cm between hearing aids
Center frequency	2.45GHz ISM	10.6MHz
Channels	80 channels	Single channel radio

Unit	COM-DEX Remote Mic	Hearing aid
Bandwidth	1MHz	660kHz (-15dB)
Duty Cycle ***	< 100 %	< 5 %
Data rate	3 Mbit/s	212kbit/s raw channel capacity
Data flow	Time division duplex (TDD)	Simplex or semiduplex
Protocol	Packet based protocol, time divided	Random access - no collision avoidance

* Bluetooth specification v 4.1 + EDR/LE published by the Bluetooth Special Interest Group (SIG)

** EIRP = Equivalent isotropically radiated power

*** Duty Cycle averaged over 1 hour of operation

Cables and transducers:

The COM-DEX Remote Mic is using a micro USB cable for charging.
Guidance and manufacturer's declaration

Electromagnetic emissions The COM-DEX Remote Mic is intended for use in the electromagnetic environment specified below. The customer or the user of a COM-DEX Remote Mic should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The COM-DEX Remote Mic uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The COM-DEX Remote Mic is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network
Harmonic emissions IEC 61000-3-2	Not applicable *)	

Emissions test	Compliance	Electromagnetic environment - guidance
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable *)	that supplies buildings used for domestic purposes.

*) Battery powered equipment
Electromagnetic immunity

The COM-DEX Remote Mic is intended for use in the electromagnetic environment specified below. The customer or the user of a COM-DEX Remote Mic should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test level	Compliance level	Electromagnetic environment - guidance
Electro-static discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.

Immunity Test	IEC 60601 Test level	Compliance level	Electromagnetic environment – guidance
Electrical fast transients/burst IEC 61000-4-4	± 2 kV for power line supplies ± 1 kV for input/output lines	Not applicable *) Not applicable *)	Not applicable *)
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	Not applicable *) Not applicable *)	Not applicable *)

Immunity Test	IEC 60601 Test level	Compliance level	Electromagnetic environment – guidance
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % UT (>95 % dip in UT) for 0.5 cycle 40 % UT (60 % dip in UT) for 5 cycles 70 % UT (30 % dip in UT) for 25 cycles <5 % UT (>95 % dip in UT) for 5 s	Not applicable *)	Not applicable *)
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment

Immunity Test	IEC 60601 Test level	Compliance level	Electromagnetic environment – guidance
----------------------	---------------------------------	-------------------------	---

NOTE UT is the a.c. mains voltage prior to the application of the test level.

*) Battery powered equipment

Electromagnetic immunity – cont.

The COM-DEX Remote Mic is intended for use in the electromagnetic environment specified below. The customer or the user of a COM-DEX Remote Mic should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test level	Compliance level	Electromagnetic environment – guidance
			Portable and mobile RF communications equipment should be used no closer to any part of the COM-DEX Remote Mic, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.

Immunity Test	IEC 60601 Test level	Compliance level	Electromagnetic environment – guidance
Conducted RF			Recommended separation distance
IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	$d = 1.2 \sqrt{P}$
Radiated RF			$d = 1.2 \sqrt{P}$ 80 MHz to 800 MHz
IEC 61000-4-3	80 MHz to 2.5 GHz	3 V/m	$d = 2.3 \sqrt{P}$ 800 MHz to 2.5 GHz

Immunity Test	IEC 60601 Test level	Compliance level	Electromagnetic environment – guidance
---------------	-------------------------	------------------	--

Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).

Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey a, should be less than the compliance level in each frequency range b.

Immunity Test	IEC 60601 Test level	Compliance level	Electromagnetic environment – guidance
			Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies
 NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the COM-DEX Remote Mic is used exceeds the applicable RF compliance level above, the COM-DEX Remote Mic should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or re-locating the COM-DEX Remote Mic.
- b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances

Recommended separation distances between portable and mobile RF communication equipment and the COM-DEX Remote Mic.

The COM-DEX Remote Mic is intended for use in the electromagnetic environment in which RF disturbances are controlled. The customer or the user of the COM-DEX Remote Mic can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the COM-DEX Remote Mic as recommended below, according to the maximum output power of the communications equipment.

**Rated maximum
output power of
transmitter
W**

**Separation distance according
to frequency of transmitter
m**

	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

Rated maximum output power of transmitter **Separation distance according to frequency of transmitter**
W **m**

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

This COM-DEX Remote Mic may be interfered with by other equipment even if that other equipment complies with CISPR emission requirements.

EMC/EMI Compliance

The COM-DEX Remote Mic complies with the following EMC/EMI standards:

Standard	Test type	Note
47 CFR Part 15, subpart C	RF emissions	USA Federal Communications Commission (FCC) requirements to intentional radiators.

Standard	Test type	Note
EN 300 328 V1.9.1	RF emissions incl. Spurious emission	EMC and radio spectrum matters for data transmission systems in the 2.4 GHz ISM band and using wide band modulation techniques
IEC 60601-1-2:2007	EMC emission Immunity, RF and ESD	Medical electrical equipment. General requirements for basic safety and essential performance. Electromagnetic compatibility.
EN 301 489-17 V2.2.1	Immunity, RF and ESD	Standard for Broadband Data Transmission Systems in the frequency range 9 kHz – 40 GHz

Directive 2014/53/EU

Widex A/S hereby declares that this COM-DEX Remote Mic is in compliance with the essential requirements and other relevant provisions of Directive 2014/53/EU.

The device contains radio transmitters operating at: 2.4 GHz, 0.9 mW EIRP.

A copy of the Declaration of Conformity can be found at:

<https://global.widex.com/doc>

N26346

Information regarding disposal

Do not dispose of hearing aids, hearing aid accessories and batteries with ordinary household waste.

Hearing aids, batteries and hearing aid accessories should be disposed of at sites intended for waste electrical and electronic equipment, or given to your hearing care professional for safe disposal.

SYMBOLS

Symbols commonly used by WIDEX A/S in medical device labelling
(labels/IFU/etc.)

Symbol Title/Description

Manufacturer

The product is produced by the manufacturer whose name and address are stated next to the symbol. If appropriate, the date of manufacture may also be stated.

Catalogue number

The product's catalogue (item) number.

Consult instructions for use

The user instructions contain important cautionary information (warnings/precautions) and must be read before using the product.

Warning

Text marked with a warning symbol must be read before using the product.

WEEE mark

“Not for general waste”. When a product is to be discarded, it must be sent to a designated collection point for recycling and recovering to prevent the risk of harm to the environment or human health as a result of the presence of hazardous substances.

Symbol	Title/Description
---------------	--------------------------

CE mark

The product is in conformity with the requirements set out in European CE marking directives.

RCM mark

The product complies with electrical safety, EMC and radio spectrum regulatory requirements for products supplied to the Australian or New Zealand markets.

FCC AND ISED STATEMENTS

FCC ID: TTY-CDRM

IC: 5676B-CDRM

Federal Communications Commission Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

NOTE:

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Changes or modifications to the equipment not expressly approved by could void the user's authority to operate the equipment.

ISED Statement / Déclaration d'ISED

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

- (1) This device may not cause interference.
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

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ISED RADIATION EXPOSURE STATEMENT:

This equipment complies with ISED RF radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

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